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EN Temporary filling

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
TO BE USED BY DENTIST ONLY

COMPOSITION: zinc oxide, zinc sulfate, calcium sulfate, excipients.

INTENDED USE

CAVIPACK is intended for temporary filling of cavities and for temporary sealing of root canal orifices during endodontic procedures. CAVIPACK is a ready-to-use paste. No preliminary preparation is required. **It does not contain eugenol.** Binds chemically when exposed to moisture (saliva). Use for class I and II cavities.

INSTRUCTIONS FOR USE

Spray the cavity with water to moisten. Place the material in the cavity with an appropriate instrument, and condense it by pressing gently against the cavity walls. Shape the filling to fit the occlusion. The surface of the material hardens within 20-30 minutes when exposed to moisture (saliva). Avoid biting for 2-3 hours. Remove from the cavity with a drill or scaler, rinse the remains with water spray.

CONTRAINDICATIONS

Do not use the product in patients with hypersensitivity to the preparation ingredients. Do not use as a temporary filling for multi-surface defects.

PRECAUTIONS

Causes serious eye damage. In case of contact, rinse immediately with plenty of water and seek medical attention. It can cause allergic skin reaction.

If swallowed, drink plenty of water, do not induce vomiting. Contact a physician for any symptoms. Use a cofferdam when working with the product. Use gloves, goggles and protective clothing.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in the original package below 25°C. Protect against moisture. After each use, close the packaging tightly. Keep out of reach of children. Expiry date is placed on the main container.

After opening the direct packaging, the shelf life does not change, provided that the packaging is ressealed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

A syringe containing 5 g of the preparation. Packaging containing 3x10 g of the preparation.

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