

EQUIA Forte™ HT

BULK FILL GLASS HYBRID RESTORATIVE SYSTEM

TECHNIQUE GUIDE



GC CORPORATION
TOKYO, JAPAN

Clinical Step by Step



1
Apply petroleum jelly
or GC Cocoa butter
inside the matrix



2
Use anatomical shaped
wedges for better
adaptation and
contact points



3
Use of tight rings
from sectional matrix
systems act as separator
of teeth to ensure good
contact points



4
OPTIONAL STEP:
Apply Cavity
Conditioner(10 sec.)
or Dentin Conditioner
(20 sec.)



5
Rinse and gently dry, do
not dessicate



6
Shake or tap
to loosen powder



7
Depress plunger.
Hold down firmly
for 2 sec.



Mix for **10 sec.**
Working time is
1 min.30 sec.
from start of mix





9

Insert on Capsule
Applier. Click twice
to prime capsule



10

IMMEDIATELY
Dispense within 10 sec.



11

Pack and contour.
Avoid moisture
contamination and
dry-out



12

Ensure complete set of
EQUIA Forte™ HT Fil and
carefully remove the ring.
Use a probe to separate
the bond between matrix
and EQUIA Forte™ HT Fil



13

Final finishing after
2 min 30 sec. from
start of mix



14

Finish the restoration
by applying the
EQUIA Forte™ coat.
DO NOT AIR BLOW







15

Light cure for **20 sec.**

Note: Steps except for 1, 2, 3 and 12 are common for Class I and II

Class II Cavity Preparation Guide

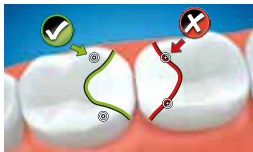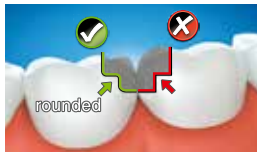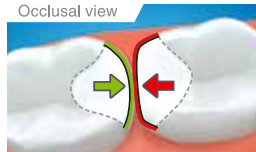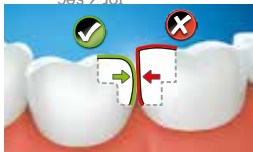

GC CORPORATION
TOKYO, JAPAN

NA8402 - 952167
30000998

