

EQUIA



1

Apply petroleum jelly or GC Cocoa butter inside the matrix



2

Use anatomical shaped wedges for better adaptation and contact points



3

Use of tight rings from sectional matrix systems act as separator of teeth to ensure good contact points



4

Apply GC Cavity Conditioner (10 sec.) or Dentin Conditioner (20 sec.)



5

Rinse and gently dry, do not desiccate



6

Shake or tap the capsule on its side to loosen the powder. Make sure to depress the plunger and hold it down for 2 seconds



7

Mix for **10 sec.**
Working time is **1 min. 15 sec.** from start of mix



8

Insert on Capsule Applicator. Click twice to prime capsule



9

IMMEDIATELY
Dispense within 10 sec.



10

Pack and contour. Avoid moisture contamination and dry-out

GC

EQUIA



11
Ensure complete set of EQUIA™ Fil and carefully remove the ring. Use a probe to separate the bond between matrix and EQUIA™ Fil



12
Final finishing after **2 min 30 sec.** from start of mix



13
Finish the restoration by applying the EQUIA™ Coat.
DO NOT AIR BLOW



14
Light cure for **20 sec.**

Note: Steps except for 1, 2, 3 and 12 are common for Class I and II

GC



951350 - NV0804
30000544