

# dmc mask

[www.dmcmask.com](http://www.dmcmask.com)

# FFP2

PROTECTIVE  
RESPIRATOR  
DM-002

MADE IN  
EUROPE



CE 0161

EN149:2001+A1:2009  
COVID-19 PPE-R/02.75-V2

2 pcs.



Extra comfort  
Non-woven fabric  
skin friendly



Easy breath  
Low respiratory  
discomfort



Perfect cover  
Fits face  
better

Partical Filtration Efficiency > 95%



4 779050 340317





[www.dmcmask.com](http://www.dmcmask.com)



### USAGE INSTRUCTIONS:


1. Before opening and touching the mask, wash your hands thoroughly with soap and water or alcohol/based hand sanitizer ( >60% alcohol content ).
2. Open the mask with both hands to the sides.
3. Fit the mask on your face covering the mouth and nose. The straps should not be between the mask and your face.
4. Pull the strap above your ears. Adjust the mask until it fits comfortably.
5. Adjust the nose bridge stripe, until it completely fits to your nose bridge.
6. Cover the mask with both hands until the mask fits tightly around the face.

### PRECAUTIONS:

1. This product is not suitable for protection against harmful gases, steam and oily aerosols.
2. The product does not generate oxygen and is not suitable for use in anoxic environments, underwater operations, during evacuation or firefighting environments.
3. It is not recommended to use this protective mask if you do not feel a good adhesion between the face and the edge of the mask during use.
4. If found during use that the mask is damaged or the breathing resistance is too large, replace with a new one.
5. This product is one-time use, masks should not be reused. Recommended use time is 4-8 hours.
6. This filtering half mask is manufactured for COVID-protection only. This filtering half mask is not a PPE device for general use and shall not be used for purposes other than protection against COVID-19.
7. This product shall not be used for more than one shift.



Techdentika Company, Tilzes st. 86, Siauliai, Lithuania  
CE notified body: INVESTIGACION DE LA INDUSTRIA TEXTIL AITEX,  
Plaza Emilio Sala 1, Spain

 20210118  
EXP 2023/01



 0161