

OptiBond™

Technique Guide

FL

Two-Component Total-Etch Adhesive

OptiBond, Unidose™, NX3, Maxcem Elite, Demi™, and Kerr are all trademarks owned by Kerr Corporation and are registered with the U.S. PTO and other countries.

fr Adhésif bicomposant à mordantage total | **es** Adhesivo grabador total de dos componentes | **de** Zweikomponenten-Adhäsiv für Gesamttätzung | **pt** Adesivo de condicionamento total de dois componentes | **it** Adesivo total-etch bicomponente | **hu** Kétkomponensű teljes maratásos ragasztó | **fi** Kaksikomponenttinen täysin etsäutuvia sidosaaine | **da** Tokomponent total ætsende adhesiv | **cs** Dvousložkové adhezivum pro celkové leptání | **bg** Двукomпонентен адхезив за тотално ецване | **pl** Dwuskładnikowy materiał łączący do całkowitego wytrawiania | **ro** Adesiv de condiționare acidă completă din două componente | **sk** Dvojsložkové lepidlo s úplným maľetraním | **hr** Dvokomponentno leplilo za potpuno jeftkanje | **sl** Dvokomponentno leplilo za popolno jeftkanje | **sv** Pålåster med tvåkomponentsbaserad Total-Etch | **tr** İki Bileşenli Toplu Aşındırma Yapıştırıcı | **el** Συγκολλητικός παράγοντας δύο συστατικών ολικής αβρόμησης | **et** Kahekomponentiline täielikult sõõvitav adhesiiv | **lt** Dvikomponentiniai klėjai „Total-Etch“ | **lv** Divu komponentu pilnīgas kodināšanas līmviela | **nl** Etskleefmiddel uit twee componenten | **uk** Двокомпонентний адхезив для техніки тотального травлення

en Store at ambient temperature | **fr** Stocker à température ambiante | **es** Guardar a temperatura ambiente | **de** Bei Raumtemperatur aufbewahren | **pt** Armazene à temperatura ambiente | **it** Conservare a temperatura ambiente | **hu** Környezeti hőmérsékleten tárolandó | **fi** Säilytetään huoneenlämmössä | **da** Opbevar ved omgivende temperatur | **cs** Uchovávejte při pokojové teplotě | **bg** Да се съхранява на стайна температура | **pl** Przechowywać w temperaturze otoczenia | **ro** A se păstra la temperatură ambientală | **sk** Skladujte pri teplote okolitého prostredia | **hr** Čuvajte na sobnoj temperaturi | **sl** Shranjujte pri sobni temperaturi | **sv** Förvaras i rumstemperatur | **tr** Ortam sıcaklığında saklayın | **el** Να φυλάσσεται σε θερμοκρασία περιβάλλοντος | **et** Säilitage keskkonnatemperatuuril | **lt** Laikyti aplinkos temperatūroje | **lv** Glabāties apkārtējās vides temperatūrā | **nl** Bewaren bij kamertemperatuur | **uk** Зберігати при температурі довкілля

fr Technologie Monomère GPD™ | **es** Tecnología del monómero de GPD™ | **de** GPD™-Monomer-Technologie™ | **pt** Tecnologia de monómero GPD™ | **it** Tecnologia a monomero GPD™ | **hu** GPD™ monomer technológia™ | **fi** GPD™-monomeeriteknologia™ | **da** GPD™-monomerteknologi™ | **cs** Monomerní technologie GPD™ | **bg** Мономерна технология GPD™ | **pl** Technologia monomeru GPD™ (GPD™ Monomer Technology™) | **ro** Tehnologie cu monomer GPD™ | **sk** GPD™ monomérová technológia™ | **hr** Tehnologija GPD™ monomera™ | **sl** Tehnologija monomera GPD™ | **sv** GPD™-monomerteknologi™ | **tr** GPD™ Monomer Teknolojisi™ | **el** Τεχνολογία μονομερούς GPD™ | **et** GPD™ Monomeeritehnoloogia™ | **lt** GPD™ monomerų technologija™ | **lv** GPD™ monomēra tehnoloģija™ | **nl** GPD™-monomeertechnologie™ | **uk** Технологія з використанням мономерів GPD™

Kerr Corporation
1717 W. Collins Ave.
Orange, CA 92667 USA
1-800-KERR-123
kerrdental.com

EC REP
Kerr Italia, S.r.l.
Via Passanti, 332
I-84018 Scalfati (SA) Italy
Tel: +39-081-850-8311

Distribution Facility
Kerr Australia Pty., Ltd.
Unit 10, 112-118 Talavera Road
North Ryde, NSW, 2113
Australia

www
Consult Instructions for Use at
www.kerrdental.com/resource-center



Rx ONLY

©2018 Kerr Corporation

DMC3319_Rev.00

OptiBond™

FL Technique Guide

Two-Component Total-Etch Adhesive

GPD™ Monomer Technology™

KAVO Kerr

OptiBond™

Technique Guide

FL

Preparation

Primer




Unidose™



Bottle

Adhesive

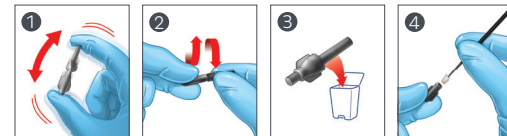



KAVO Kerr

OptiBond™

Technique Guide

FL

Direct Restorations

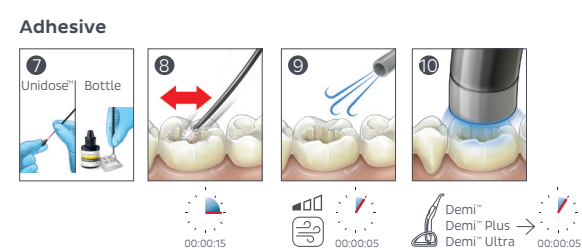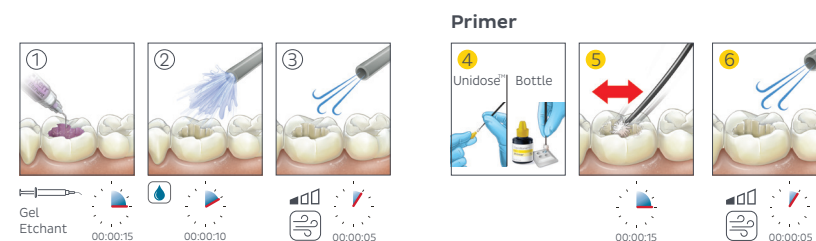

KAVO Kerr