

Descaling of POWER steamer 1+2



Please note!

- ☑ Rinse the unit with water at regular intervals, or weekly if using a salt-based softening system. For this purpose, also observe the instructions in the operating manual.
- ☑ Only use the Renfert descaling solution „POWER steamer descaler“ (provides corrosion protection).
- ☑ Other descaling solutions like citric acid could damage the unit.
- ☑ Caution: The descaling solution is corrosive!
- ☑ Wear protective goggles and protective gloves during descaling.
- ☑ Do not operate the unit during descaling!
- ☑ The inlet must remain open during the entire descaling process.

Step-by-step

1. Disconnect the unit and allow it to cool down.
2. Open the service opening.
3. Rinse the pressure tank with water and a cleaning brush.
4. Close the service opening.
5. Fill the pressure tank with 1l of water, then add 1l Renfert POWER steamer descaler.
6. Add another 0,6–0,8l of water after 15 minutes.
If foam develops, only continue to fill after the foam has subsided.
7. Allow the descaling solution to soak in for 2–4 h for lighter soiling and 8–14 h for heavier soiling.
8. Open the service opening and drain the descaling solution.
9. The used descaling solution may be disposed of with wastewater if heavily diluted.
10. Rinse the pressure tank several times with water and brush clean.

Further information

Further information such as instructions, safety data sheets and application videos, etc. can be found on our website.



[www.renfert.com/
power-steamer-1](http://www.renfert.com/power-steamer-1)



[www.renfert.com/
power-steamer-2](http://www.renfert.com/power-steamer-2)

