

Noritake

CLEARFIL MAJESTY[™] ES Flow Universal Strong. Universal. Flowable.

ELAND MARTI

Your new go-to composite



Intuitive shade matching goes flowable

CLEARFIL MAJESTY[™] ES Flow Universal

into an effortless endeavor-no shade guide required! With just two complementary shades, U and UD, it seamlessly blends with the surrounding tooth structure, empowering you to create beautiful restorations with minimal effort. As it is as strong as many paste-type composites, it opens the door to a wide range of applications.



1 Shade

Covers all 16 VITA* shades in all cavities except those lacking of a back wall and of shade B3 and darker

2 Flowabilities

Impressive Strength

Sufficient for use even posterior restorations

Areas of application

CLEARFIL MAJESTY™ ES FLOW Universal

- Direct restorations for all cavity classes, cervical lesions (e.g. root surface caries, v-shape defects), tooth wear and tooth erosion
- Cavity base/liner
- Corrections of tooth position and shape
- Intraoral repairs of fractured restorations
- Cementation of ceramic and composite inlays, onlays and veneers with less than 2 mm thickness



SCAN FOR MORE INFORMATION ABOUT CLEARFIL MAJESTY[™] ES Flow Universa





Important part of the Universal Excellence portfolio

At Kuraray Noritake Dental Inc., we understand the demands of modern dental practices. Balancing complex procedures, tight schedules, and patient care is no easy task. That is why we have spent years refining our materials—so you can focus on what matters most: your patients. Our universal products simplify your workflow, offering you high-quality materials and tools that work seamlessly with your expertise.

CLEARFIL MAJESTY[™] ES FLOW Universal is the latest resin-based addition to our universal portfolio. Being suitable for a wide range of applications within direct and indirect restorative workflows, it is a valuable complement to existing paste-type composites and resin cements.



Great mechanical properties

CLEARFIL MAJESTY™ ES Flow Universal

Low

- Flexural strength 151 MPa
- Compressive strength 373 MPa
- Filler load 75 wt/% / 59 vol%
- ✓ Radiopacity 140 % AI

Data source Kuraray Noritake Dental Inc.

Shift in flowable composites

From temporary fixes to permanent solutions

By integrating high filler loading (75, resp. 78 wt/%) and achieving over 150 MPa flexural stength, CLEARFIL MAJESTY[™] ES Flow Universal challenges the traditional view of flowable composites as merely temporary fixes. Instead, it positions itself as a permanent solution capable of delivering both aesthetic excellence and structural reliability.

Simplified shade concept	\rightarrow	Fe ha
Two flowability options with excellent handling	\rightarrow	No ine
High mechanical strength	\rightarrow	E\ (o
Easy polishing and long-lasting gloss	\rightarrow	Ti
Clever syringe design	\rightarrow	Re

Super Low

- ✓ Flexural strength 152 MPa
- V Compressive strength 374 MPa
- ✓ Filler load 78 wt/% / 60 vol%
- ✓ Radiopacity 150 % AI

ewer work steps and reduced assle of inventory management

on-sticky composite for a wide range of dications offering easy dispensing and sculpting

ven suitable for stress-bearing areas cclusal surfaces)


me-saving procedure and high aesthetics

educes the risk of air bubbles, no dripping, asy modelling with syringe tip

Simplified shade concept

Two shades do the trick

- CLEARFIL MAJESTY[™] ES Flow Universal is a flowable composite that consists of just two shades:
- One single universal shade (U) for all posterior restorations
- Two shade options universal (U) and universal dark (UD) for the anterior region.

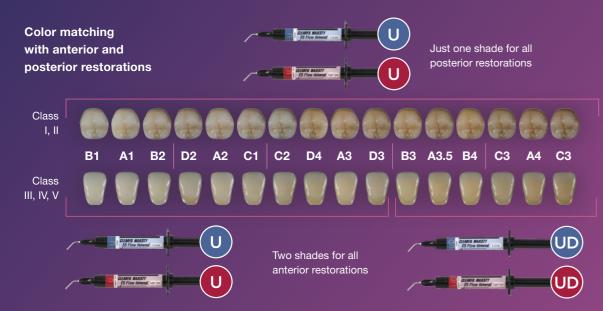

Wonder how it is possible to achieve brilliant aesthetics with this slim shade range?

A smart combination of an enamel-like translucency, dentin-like chroma and Light-Diffusion Technology (LDT) does the trick. Thanks to the enamel-like translucency, light is transmitted through the material, bringing out the colour of the surrounding tooth structure. Fine-tuning of the chroma, which is not affected by the surrounding tooth structure, supports natural outcomes. Finally, LDT makes restorations diffuse light



in a similar way as enamel and dentin. As a consequence, the material integrates easily and smoothly into the surrounding tooth structure.

This works extraordinarily well whenever several cavity walls are still present, like in posterior cavities (Class I or II) and cervical lesions (Class V). Light is simply reflected, diffused in the desired way and the original tooth shade is revealed. That is why in these cavity classes, a single shade (U) is sufficient. Cavities in darker teeth with a low amount of surrounding tooth structure that could be reflected and brought out (Class III and IV cavities), however, benefit from the use of a second, darker shade (UD).





3M ESPE Filtek

Supreme Ultra

Flowable



CLEARFIL MAJESTY™ ES Flow Universa (highest level of light diffusion)

Ivoclar Vivadent Tetric EvoFlow

ON PAPER

CLEARFIL MAJESTY™ ES Flow Universa (highest level of light diffusion)

OFF PAPER



Ivoclar Vivaden Tetric EvoFlov



Every need covered

CLEARFIL MAJESTY[™] ES Flow Universal is available in two different levels of flowability: Low and Super Low. While Low is a standard flowable composite suitable for a wide range of indications, the Super Low option is special in that it retains the given shape. Hence, it is ideal for building posterior cusps and ridges, making it the go-to material for use in occlusal surface areas. No matter which flowability is selected, the material is easy to extrude. Due to its low stickiness, handling and sculpting with the syringe tip is also easy.



Clever syringe design: Effortless, accurate application

The syringe of CLEARFIL MAJESTY™ ES Flow Universal is designed to support effortless and accurate application. The needle tip design avoids air bubble formation and dripping after use. Moreover, modeling with the tip is easy, while the plunger reveals the shade and offers an appropriate extrusion force.



Source: Kurarav Noritake Dental Inc




*Not a trademark of Kurarav Noritake Dental Inc

Performance you can trust

High mechanical strength: **Resistant to occlusal forces**

Early flowable composites had a major drawback: their weak mechanical properties set limits to their use or required a final layer of stronger composite on top. CLEARFIL MAJESTY™ ES Flow Universal is different its flexural and compressive strength are similar to those of conventional, paste-type composite systems. Consequently, there are no limitations regarding its use in stress-bearing areas like the occlusal surface of posterior teeth. This simplifies clinical procedures and provides for stable outcomes even in the long term.

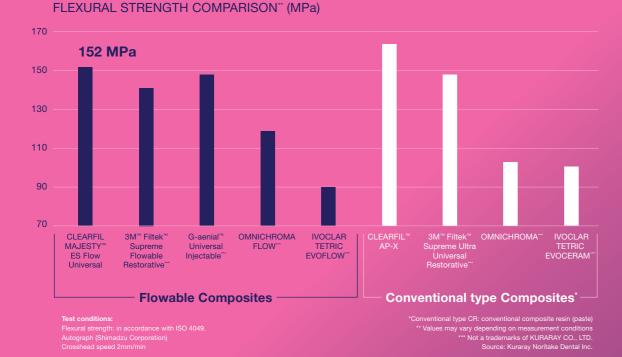


Suitability for flowable injection technique: quicker restoration of multiple teeth

Restoring multiple teeth with composite in the conventional way can be laborious and unpredictable. Creating a wax-up, producing a transparent silicon index and injecting a suitable flowable composite to build up the teeth to the desired shape can speed up the clinical procedure and support predictable outcomes. CLEARFIL MAJESTYTM ES Flow Universal can be used for this workflow due to its wellbalanced flowability and mechanical properties. It easily flows into every spot of the index and maintains a stable, smooth surface over time after polishing.

Easy polishing: Natural beauty ideally supported


Like in other composites of the CLEARFIL MAJESTY[™] ES series, CLEARFIL MAJESTY[™] ES Flow Universal contains submicron fillers and cluster fillers developed by Kuraray Medical Inc. These fillers are so small that polishing is easy and quick, while light is ideally reflected for a natural effect. The small particle size also prevents the formation of large irregularities on the surface of the cured product after toothbrush abrasion. Hence, the natural gloss may be expected to last.

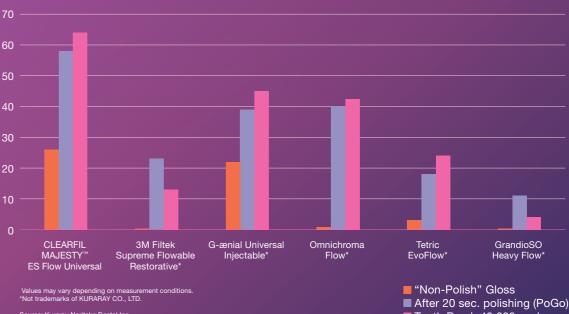

Source: Kurarav Noritake Dental Inc

GLOSS LEVEL (%)

test conditions: Specimen o10 mm, thickness 2 mm. Surface roughened with #600 water-resistant abrasive paper Gloss level of all the materials was measured below the conditions: Non-polish: After ethanol wiping Polishing: Polished with composite resin finishing points PoGo (Dentsply Sirona) at 5000 rpm. Toothbrush wear: 40,000 strokes were worn by the toothbrush at a load of 250 g, 30 strokes/minute.

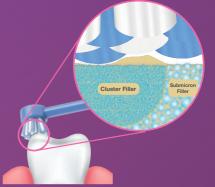

Tooth Brush 40,000 cycles

Clinical procedures: simple steps

Direct restoration with CLEARFIL MAJESTY[™] ES Flow Universal







1. Prepare the cavity and 2. Dispense and sculpt apply bonding.

3. Light-cure CLEARFIL™ MAJESTY ES Flow.

4. Finish & Polish

Flowable injection technique with **CLEARFIL MAJESTY[™] ES Flow Universal**






enamel layer on the surface of the teeth.

silicone index produced over the wax-up to check its fit. Injection channels integrated with a flowable syringe

tip are visible.

each side

adjacent teeth to protect Syringe to the entire surface of every other tooth, then rinse and dry.

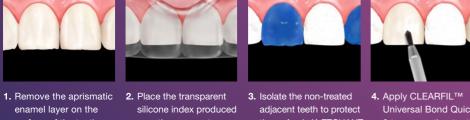





5. Light-cure the adhesive



6. Reposition the silicone 7. Light-cure the flowable 8. Remove the silicone index in the patient's mouth and inject CLEARFIL™ MAJESTY FS Flow Universal through the injection channels of the pre-treated teeth.



composite through the index, the tape used transparent index from for protection and separation, and remove excess with a blade



9. Repeat the procedure for the previously untreated teeth (steps 3 to 8).



10. Polish with burs and strips for interproximal area. Use TWIST DIA™ for Composite to finish polishina.*





KATANA[™] Cleaner - Optimising the bond

KATANA[™] Cleaner removes contamination to optimise adhesive procedures. It has a high cleaning effect due to the surface-active characteristic of MDP Salt. Suitable for intra- and extra-oral use, it is the optimal solution for the cleaning of all bonding surfaces, which are potentially contaminated with saliva or blood for example, after tooth preparation.

All indications, one bond

CLEARFIL[™] Universal Bond Quick 2 is the innovative universal bonding that works immediately: apply, dry, and cure. No need to wait, rub in thoroughly, or apply several coats. The universal adhesive provides for reliable and durable bonding to tooth structure and composite as well as all other direct and indirect restorative materials. The product may be used as desired: self-etch, selective-etch, or total-etch. The viscosity and strength of the bonding laver have been optimized for easier application, procedural safety and simplicity, and a great long-term performance.

K-ETCHANT Syringe - Etching made easy

K-ETCHANT Syringe contains 35% phosphoric acid and offers thixotropic properties. With the new finer (0.5 mm) 'needle tip', the etchant can be applied with great precision, while thanks to thixotropy it remains exactly where it is applied. This gives you full control over the application.

TWIST[™] DIA for Composite - Creating natural gloss*

TWIST[™] DIA for Composite is indicated for polishing all kinds of composites. The "pre-polisher" creates the final form of the restoration from any angle imaginable. With the "high-shine polisher", a natural high-gloss result is obtained within seconds. TWIST™ DIA for Composite is autoclavable and thus reusable. The combination of flexible components and diamond particles delivers an unprecedented high-gloss result with each use.

CARIES DETECTOR - The caries revealer

CARIES DETECTOR plays a major role in maintaining pulp vitality and

preserving tooth structure by enabling you to clearly distinguish between caries-infected, and caries-affected, healthy dentin. As only the irreversibly infected dentin is stained red, you can confidently remove this tissue, while leaving the healthy dentin in place.

CLEARFIL[™] Universal Bond Quick 2 -



Ordering Information

CLEARFIL MAJESTY[™] ES Flow Universal

#4600-EU	Low (U)	1 syringe 1.5 ml x 1, needle tip x ⁻
#4601-EU	Low (UD)	1 syringe 1.5 ml x 1, needle tip x ⁻
#4620-EU	Super Low (U)	1 syringe 1.5 ml x 1, needle tip x ⁻
#4621-EU	Super Low (UD)	1 syringe 1.5 ml x 1, needle tip x $^{-1}$

#3331-EU 20x Needle Tip (N)



EU Importer Kuraray Europe GmbH

Philipp-Reis-Strasse 4, 65795 Hattersheim am Main, Germany Phone +49 (0)69 305 35 835 Fax +49 (0)69 305 98 35 835 www.kuraraynoritake.eu centralmarketing@kuraray.com

Before using this product, be sure to read the Instructions for Use supplied with the product.
The specifications and appearance of the product are subject to change without notice.
Printed colour can be slightly different from actual colour.

"CLEARFIL" and "CLEARFIL MAJESTY" are registered trademarks or trademarks of KURARAY CO., LTD

15 pcs 15 pcs 15 pcs 15 pcs





