



### Components

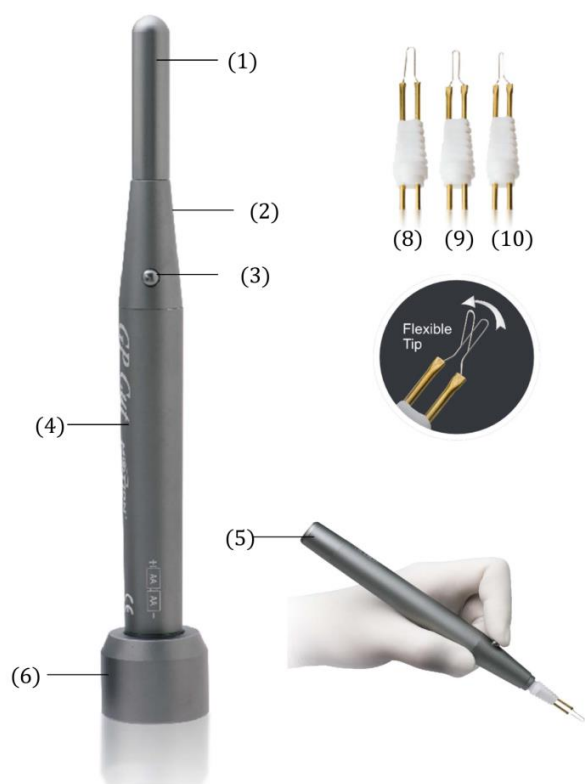

- (1) Cap
- (2) Front
- (3) Power Button
- (4) Handle
- (5) Battery cover
- (6) Stand Base
- (7) Two AA batteries (not shown)
- (8) Large Tips (2.0 mm)
- (9) Medium Tips (1.2mm)
- (10) Small tips (0.5mm)

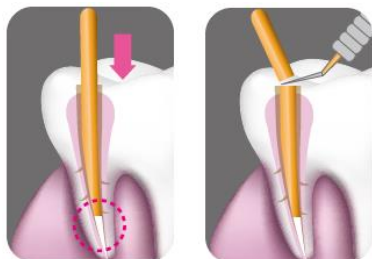

### Specification

Dimension (LxW)	20cm x 1.85 cm (tips include)
Voltage	2 AA batteries (DC 3V)
Tip Type	3 Tips (L: 2.0mm; M: 1.2mm; S: 0.5mm)
Temp. Reach	Over 500 degree
Cool down period	less than 1 second
Stand base size	3 cm x 2.5 cm

### GP CUT Feature

- One button only, two batteries power supply, easy use design.
- Three tip size available.
- One second hip up to 500 degree high temperature.
- Fast tip cool down design, safety use.
- Flexible tip design, cutting smoothly.

### Simple to Use

- Install two AA batteries according to the diagram on GP Cut body. For best results, use alkaline batteries.
- Insert the selected tip on the front of GP Cut. Cutting wire may be bent as needed for access.
- Contact gutta percha at desired cutting point and press power button for 2 secondds. Cut gutta percha and release power button.
- for maximum patient safety, wait 3 seconds before withdrawing GP Cut from the oral cavity.
- Body or GP Cut may be surface sisinfected or covered with plastic barrier (sold separately) before use.
- Interchangeable cutting tips may be cols disinfected only.
- Replace batteries whtn unit stops functioning properly.

### CAUTION:

- (1) DO NOT touch tip during operation due to high temperature.
- (2) DO NOT press power button over 5 seconds.
- (3) NO water sprays on unit.

The device has been tested to meet the following standards: IEC 60601-1, IEC 60601-1-2, 93/42/EEC, and ISO 13485.

### Manufacturer:

#### Motion Dental Equipment Corp.

Address: 6F-2 & -3, No. 27, Lane 61, Sec. 1, Guangfu Rd, Sanchong District, New Taipei City, Taiwan (R.O.C)

TEL : +866-2-2278-2199 FAX: +886-2-8511-2492

Website: <http://www.motiodental.com> E-mail: [motion@twmotion.com](mailto:motion@twmotion.com)