


3M Science.
Applied to Life.™



3M™ Astringent Retraction Paste

Retract tissue gently.

A great impression – even a digital impression – starts with controlling tissue and moisture. Our capsule's extra-fine tip delivers hemostatic paste right into the sulcus to gently retract tissue and control bleeding.¹

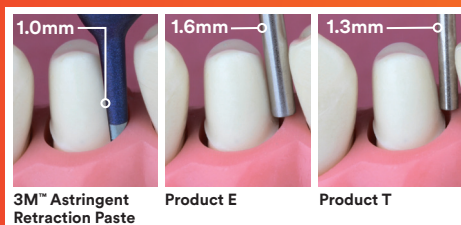

3M™ Astringent Retraction Paste

Product E

Product T



And save time.

Unlike gingival retraction cords that can be difficult and time-consuming to place, the 3M astringent retraction paste is a fast, convenient and effective solution for gingival retraction. With its extra-fine tip, the 3M astringent retraction paste delivers a 15% aluminum chloride astringent paste right into the sulcus, pushing the gingival tissue away from the tooth and allowing you to capture a detailed impression of the preparation margin.



Indications

3M™ Astringent Retraction Paste is indicated for the temporary tissue retraction of the marginal gingiva to provide a dry sulcus when the periodontium is healthy, such as:

- ▶ Taking impressions (material-based or digital)
- ▶ Preparation of temporary casts
- ▶ Preparation of Class II and V fillings

The product must not be used in patients suffering from a diseased periodontium, open furcations or exposed bone.


The retraction paste can be used with or without cord.



Simple steps to a clean and dry working field



The 3M™ Astringent Retraction Paste fits most composite dispensers. Extrude a small amount of paste and discard.

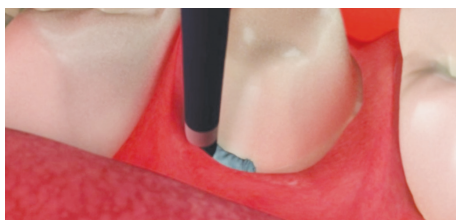

Insert retraction paste tip into the sulcus. The tissue is mechanically retracted.



Slowly and steadily, inject astringent retraction paste into the sulcus. Completely fill the sulcus.



Optional: Procedure with cords

For greater gingival deflection, the astringent retraction paste can be used in combination with retraction cords.



Leave astringent retraction paste on for a minimum of 2 minutes.



Completely remove astringent retraction paste with air-water spray and suction.

Ordering information

Item #	Product information
56944	3M™ Astringent Retraction Paste 25 Capsules Refill Pack
56945	3M™ Astringent Retraction Paste 100 Capsules Value Pack

Note: The content of one unit-dose capsule is sufficient to treat up to three (3) teeth. The capsules are for single use and must not be used on more than one patient.



No dedicated dispenser needed. The unit-dose capsule fits most composite dispensers.

3M Oral Care

3M United Kingdom PLC
Charnwood Campus
10 Bakewell Road
Loughborough
Leicestershire LE11 5RB
0845 602 5091

3M Ireland Limited
The Iveagh Building
The Park
Carrickmines
Dublin 18, Ireland
1 800 320 500

1 3M astringent retraction paste tip size comparison. 3M Oral Care internal data. Claim number 04915.

GD022. J393262.
3M is a trademark of the 3M company.
© 3M 2018. All rights reserved.

