



ORO CLEAN® Liquid

Strong cleaner to prevent biofilm



- ▶ **Strong cleaning power**
- ▶ **Gentle to materials**
- ▶ **Non-foaming**

ORO CLEAN® Liquid is a highly effective concentrate for the cleaning of suction units, spittoon bowls, and amalgam separators. Thanks to a carefully balanced combination of active agents, the product has excellent cleaning properties and removes even persistent residue such as scale deposits, blood and other build up. ORO CLEAN® Liquid is free of aldehydes and alcohols, making it suitable for a wide range of materials. ORO CLEAN® Liquid's non-foaming action is non-damaging to the suction pump.

Use



For cleaning of suction units, use an aqueous solution and dosage as required. To measure the correct dosage, use lid as a measuring cup (lower scale = 20 ml, upper scale = 40 ml). Rinse with water prior to cleaning. At the end of each working day and as required, aspirate 1.5 liter of working solution per unit using the OROCLEAN® Cup. Pour 0.5 liter down the spittoon bowl. After cleaning, rinse the unit thoroughly with water. In addition, rinse the unit thoroughly with water after surgical / invasive procedures.

Scope of Application



- ▶ For the cleaning of suction units, spittoon bowls, and amalgam separators

Precautions



R36 / 38 Irritating to eyes and skin. S2 Keep out of the reach of children. S26 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. S28 After contact with skin, wash immediately with plenty of water. S37 / 39 Wear suitable gloves and eye / face protection. S45 In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible). Always read label and product information before use.

Delivery Forms



2 liter, 5 liter

Dosing

		+	
	Concentrate		Water
2%	40 ml	+	1960 ml

Composition in 100 g

Alkaline cleaning compounds, Antifoaming agents, Special surfactants.

