

Indication 1 Cementation of Crowns

PANAVIATM V5

Silica-Based Ceramics (e.g. Lithium Disilicate)


Clean and dry the tooth surface in the usual manner. As necessary, trial fit the crown using the Try-in Paste, wash and remove.

1 Apply hydrofluoric acid or K-ETCHANT Syringe, rinse and dry.

2 Apply CERAMIC PRIMER PLUS and dry.

3 Apply Tooth Primer, leave for 20 sec., and thoroughly dry with mild air.

4 Dispense cement and place the crown.

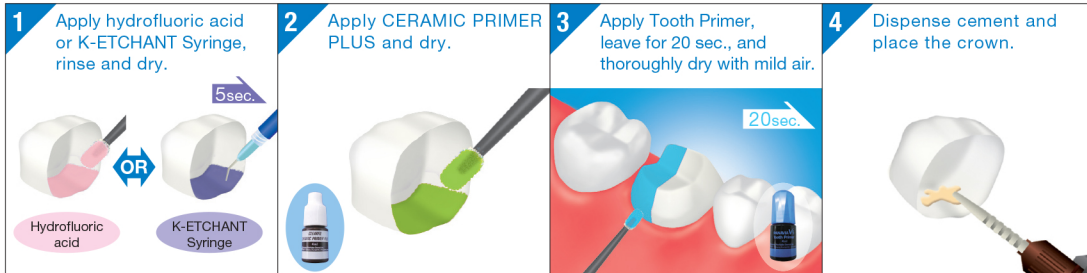

5 Remove the excess cement using either method.

Opaque shade is used for method B.

A. Tack-curing

① Light-cure for 3 to 5 sec.


② Remove with a dental explorer.

B. Using a small brush

① Remove with a small brush.

② Light-cure margins. Refer to table 2.

6 Maintain isolation for 3 minutes, or light-cure* for translucent crowns.

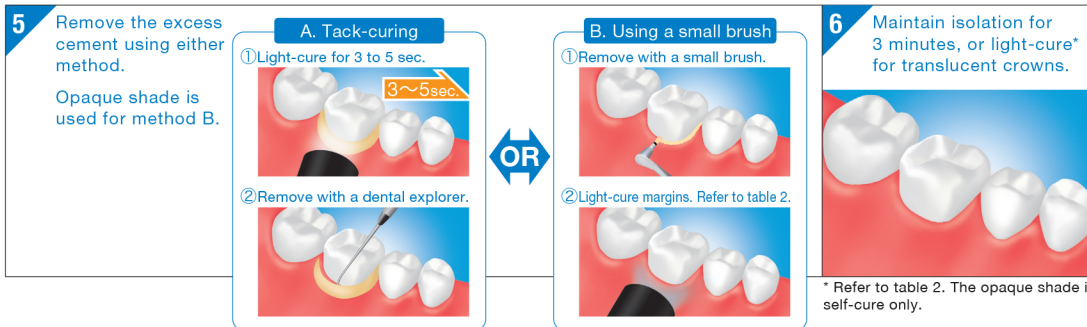

* Refer to table 2. The opaque shade is self-cure only.