


# Indication 1 Cementation of Crowns

PANAVIA<sup>TM</sup> V5

## Zirconia

Clean and dry the tooth surface in the usual manner. As necessary, trial fit the crown using the Try-in Paste, wash and remove.

- 1** Blast with alumina powder, then ultrasonic clean and dry.

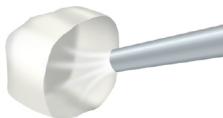

- 2** Apply CERAMIC PRIMER PLUS and dry.



- 3** Apply Tooth Primer, leave for 20 sec., and thoroughly dry with mild air.



- 4** Dispense cement and place the crown.



- 5** Remove the excess cement using either method.

Opaque shade is used for method B.

### A. Tack-curing

- ① Light-cure for 3 to 5 sec.



- ② Remove with a dental explorer.



### B. Using a small brush


- ① Remove with a small brush.



- ② Light-cure margins. Refer to table 2.



- 6** Maintain isolation for 3 minutes, or light-cure\* for translucent crowns.

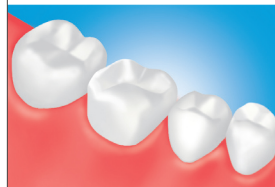

\* Refer to table 2. The opaque shade is self-cure only.