


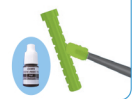

Indication 5 Cementation of Posts

PANAVIA V5

After preparation of the cavity and trial fit of the post.

1 Treatment of the post		2 Apply Tooth Primer, leave for 20 sec.	3 Remove any excess liquid with a paper point, and thoroughly dry with mild air.
<p>Glass-fiber and ceramic posts</p> <p>① Apply K-ETCHANT Syringe, leave for 5 sec., rinse and dry.</p> <p>② Apply CERAMIC PRIMER PLUS and dry.</p> 	<p>Metal posts</p> <p>① Blast with alumina powder, then ultrasonic clean and dry.</p> <p>② Apply CERAMIC PRIMER PLUS and dry.</p> 		
4 Apply the mixed paste over the post or the entire tooth surface*.	5 Place the post quickly, by slightly vibrating it.	6 Spread the excess cement over the coronal base and post head, then light-cure*.	7 Place the core buildup composite resin*.
			

* Refer to table 1 for Working time

* Refer to table 2. For the opaque shade, set for 3 minutes after placement of the post.

* Prepare the abutment tooth 6 minutes after seating the dental post.