
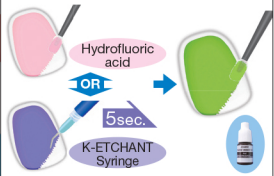










# Indication 2 Cementation of Veneers

## Silica-Based Ceramics (e.g. Porcelain)

Clean and dry the tooth surface in the usual manner.

<p><b>1</b> As necessary, trial fit the veneer using Try-in Paste, wash and remove.</p> 	<p><b>2</b> ① Apply hydrofluoric acid or K-ETCHANT Syringe, rinse and dry. ② Apply CERAMIC PRIMER PLUS and dry.</p> 	<p><b>3</b> ① Apply K-ETCHANT Syringe to the enamel, leave for 10sec., rinse and dry.</p> 	<p>② Apply Tooth Primer, leave for 20 sec., and thoroughly dry with mild air.</p> 
<p><b>4</b> Dispense cement and place the veneer.</p> 	<p><b>5</b> Remove the excess cement using either method.</p> <p>Opaque shade is used for method B.</p> <p><b>A. Tack-curing</b> OR <b>B. Using a small brush</b></p> <p>① Light-cure for 3 to 5 sec.</p>  <p>② Remove with a dental explorer.</p>  <p>① Remove with a small brush.</p>  <p>② Light-cure margins. Refer to table 2.</p> 		<p><b>6</b> Maintain isolation for 3 minutes, or light-cure* for translucent veneers.</p> 

\* Refer to table 2. The opaque shade is self-cure only.