



FAST DURABLE COMFORT



# Ufi Gel SC

PERMANENTLY SOFT RELINING IN ONE APPOINTMENT

# Ufi Gel SC

## CHAIRSIDE RELINES THAT ARE FAST, RELIABLE AND EFFECTIVE

### Indications

**Ufi Gel SC** is indicated for denture patients with insufficient fit or adhesion of the denture, flabby ridge, pressure spots, mucosa- and jaw inflammation, new adaptation of dentures etc. It can be used on almost all cold-curing denture materials.

### Advantages at a glance

- Complete relining chairside in one appointment
- Extremely high adhesion with special adhesive
- Biocompatible (no methacrylates), neutral in taste and odor
- Color-stable and esthetic, with chameleon effect



### Ufi Gel SC – a speciality material


The adhesive (patent pending) provides extremely good connection between the relining and the denture. The adhesive has to act on the denture only 60 seconds before the relining material can be applied.

**Ufi Gel SC** is an A-silicone with a low shore-A hardness and remains permanently soft. Masticatory forces are well absorbed and distributed over a large area.

**Ufi Gel SC** is biocompatible and does not release any plasticizers or monomers, etc. after curing. The material does not discolor, remains permanently neutral in taste and is easy to clean.

**Ufi Gel SC** is light-pink and transparent. This produces a natural shade adaptation of the relining material to the denture (chameleon effect). Even on older dentures **Ufi Gel SC** adapts optimally in shade.

The adhesive, the material properties of the A-silicone and the application from the cartridge allows for the relining of a denture in one session.

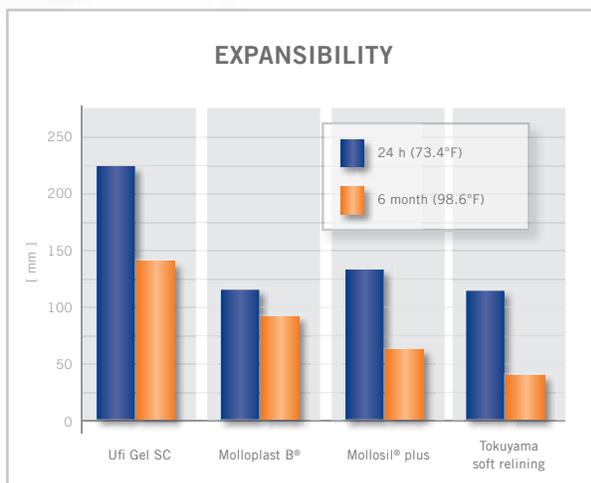

D. Welker et al.; Comparative material study of a soft synthetic material, clinic of Friedrich-Schiller-University Jena, March 1998.

Voco is not the holder of mentioned trademarks.

# Ufi Gel SC

## EASY CHAIRSIDE RELINING IN ONE APPOINTMENT...

Apply a thin layer of adhesive evenly on prepared surfaces.

approx. 2 min.

Apply Ufi Gel SC

approx. 1 min.



After preparing the denture (grinding and cleaning), apply the adhesive




Apply Ufi Gel SC

Insert denture into patient's mouth after 1 minute

approx. 7 min.

10 minutes after taking out remove excess material and finish rims

approx. 12 min.

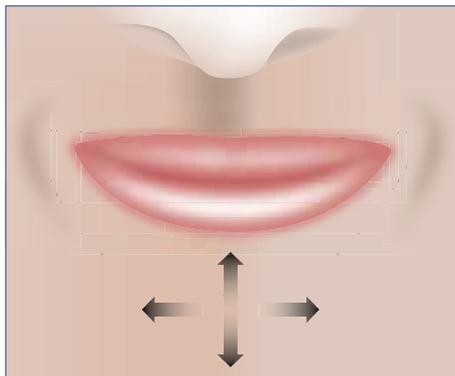

Close mouth for 1 minute and then carry out masticatory movements for 5 minutes



Precision work with the enclosed Lisko polishing disk

Apply glazing and let it set for 10 minutes

approx. 12 min.

Approx. 34 minutes net working-time



Mix Glazing base and catalyst homogeneously in a 1:1 ratio



Finished relined denture

## Ufi Gel Reline Products

RELINING MATERIALS WITH MORE THAN 10 YEARS OF CLINICAL SUCCESS



A-silicone based permanent soft denture relining material

Direct hard relining material in an automix cartridge for total and partial dentures, denture repairs as well as lengthening of denture rims

### Presentation

#### Ufi Gel SC

- REF 2041 kit – cartridge 50 ml Ufi Gel SC, adhesive 10 ml, each 10 ml glazing base/catalyst, mixing tips type 8, accessories
- REF 2040 refill – cartridge 50 ml, mixing tips type 8
- REF 2029 refill 10 ml Ufi Gel SC adhesive
- REF 2047 refill each 10 ml glazing base/catalyst
- REF 2187 refill 50 mixing tips type 8 (pink)

#### Trimming disk:

- REF 2049 refill trimming disk, 10 pieces

#### Ufi hard C

- REF 2215 kit – cartridge 80 g, adhesive 10 ml, mixing tips type 8
- REF 2218 intro kit – cartridge 80 g, dispenser type 2, adhesive 10 ml, mixing tips type 8
- REF 2216 refill – cartridge 80 g, mixing tips type 8
- REF 2217 refill 10 ml adhesive
- REF 2187 refill 50 mixing tips type 8 (pink)

VOCO America Inc.  
1245 Rosemont Drive  
Suite 140  
Indian Land · SC 29707

www.vocoamerica.com  
infousa@voco.com

Toll-free phone: 1-888-658-2584  
Fax: 1-888-849-3989

