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User instructions

Dear user,

KaVo hopes that you enjoy your new high-quality product. Following the instructions below will allow you to work smoothly, economically and safely.

Symbols

	See the section Safety / Warning Symbols
	Important information for users and technicians
	CE mark (Communauté Européenne). A product with this mark meets the requirements of the applicable EC directive.
	Action request

User group

The instructions for use are for dentists and dental assistants, as well as hygiene personnel in the dental practice.

Purpose

KaVo CLEANspray: Cleaning spray for validated intensive internal cleaning of KaVo straight and contra-angle handpieces and KaVo turbine handpieces.

KaVo CLEANspray is free from alcohol (non protein-fixing) and is designed for the manual intensive interior cleaning of handpieces/contra-angle handpieces and turbines (incl. spray channels) in accordance with DIN ISO 17664. KaVo CLEANspray is a medical device according to relevant national statutory regulations. The suitability for cleaning has been proven according to the series of standards DIN ISO 15883 and according to the specifications of the German Society for Sterile Supply e.V. (DGSV) and has been confirmed via an expert's report (www.kavo.com). This confirmation can also be transferred to instruments and turbines of former series since they are similar in design.

KaVo DRYspray: Drying spray for validated intensive internal cleaning of KaVo straight and contra-angle handpieces and KaVo turbine handpieces.

KaVo DRYspray is used directly after the use of KaVo CLEANspray for drying air, water and gear channels.

Product description

Order dates

REF	Designation	Consisting of
1.007.0573	CLEANSpray- / DRYspray Starterset 2116 P	1 can of KaVo CLEANspray 1 can of KaVo DRYspray 1 adapter set INTRA + MULTIflex
1.007.0579	Pack of KaVo CLEANspray 2110 P	4 cans of KaVo CLEANspray
1.007.0580	Pack of KaVo DRYspray 2117 P	4 cans of KaVo DRYspray
1.007.1776	Spray attachment INTRA	
1.007.1775	Spray attachment for KaVo MULTIflex turbines	


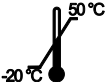
NOTE

Please note that these spray attachments are designed to be reused with additional CLEANspray/DRYspray cans and are not to be disposed with the empty cans.

Transportation and storage conditions

Observe the instructions for storage and handling on the can and packaging.

	Content
	Storage and transportation temperature -20°C – +50°C
	Humidity 5% - 85%
	CE mark
	Expiration date YYYY-MM
	Production batch Xxxxxx
	Please note the instructions for use
	EEC sign for pre-packed products
	Symbol indicating compliance with Directive 75/324/EEC for aerosol dispensers (inverted epsilon)

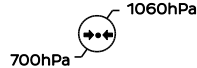 The image shows two rectangular boxes. The left box contains the number '650' and the right box contains the number '405'. Between the two boxes is the word 'or'.	Total capacity (in mL) is the total volume of the open container (up to the brim) of the compressed gas dispenser
	Safety-related information
	Protect product from exposure to sunlight
	Air pressure restrictor
	Store in a dry location

The products may only be used at room temperature (18 °C – 25 °C).

Operation

Use KaVo CLEANspray

- ▶ Place the appropriate spray attachment on the can.
- ▶ Place the instrument on the spray attachment.
- ▶ Operate the spray with 3 spray jets during 2 seconds each in the instrument.
- ▶ Remove the instrument from the spray attachment.
- ▶ Lay the instrument down, and let the cleaner work for 1 minute. We recommend an exposure time of 8-10 minutes for first use.

Use KaVo DRYspray

- ▶ Place the appropriate spray attachment on the can.
- ▶ Place the instrument on the spray attachment.
- ▶ Apply spray into the instrument for at least 3 seconds.
- ▶ Remove the instrument from the spray attachment.

**NOTE**

Keep bottle upright during use.

**NOTE**

After this intensive inner cleaning, perform the additional preparation steps according to the valid recommendations and guidelines.

If the instruments and turbines are disinfected in the thermodisinfectant, they must be treated afterwards with products from the KaVo care system (KaVo Spray/ROTAspray/QUATTROcareplus spray). If the instruments are sterilised in an autoclave, they must be cared for beforehand. Observe the care instructions in the instructions for use of the corresponding instruments and turbines.

KaVo CLEANspray:

Hazard Statements + Precautionary statements:

Warning: H229: Pressurised container: May burst if heated.
P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P251: Do not pierce or burn, even after use.
P410+P412: Protect from sunlight. Do not expose to temperatures exceeding 50°C / 122°F

KaVo DRYspray:

Hazard Statements + Precautionary statements:

Warning: H229: Pressurised container: May burst if heated.
P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P251: Do not pierce or burn, even after use.
P410+P412: Protect from sunlight. Do not expose to temperatures exceeding 50°C / 122°F.

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 The image shows two rectangular boxes. The left box contains the number '650' and the right box contains the number '405'. Between the two boxes is the word 'of'.	Totale inhoud (in ml) is de strijkvolle inhoud van de open bus van de spuitbus
 A black warning symbol consisting of a triangle with an exclamation mark inside.	Veiligheidsgerelateerde informatie
 A black symbol representing sun protection, showing a sun with rays and a small square below it.	Product beschermen tegen zonnestrallen
 A black symbol for pressure limits, showing a circle with two arrows pointing left and right, and two lines extending from the circle to the text '1060hPa' and '700hPa'.	Luchtdruk begrenzing
 A black symbol for dry storage, showing an umbrella with raindrops falling from it.	Droog bewaren

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